


## Model: ZORALM001

Thank you for purchasing the ZORALM001 Fever Detection System. Please take a moment to review the setup and operational information here.


### Initial Setup

The ZORALM001 is a complete, “Stand Alone” unit. Simply unbox the unit, remove all packing material and plastic protectors; then plug the unit into the wall using the provided power connection cable. The ZORALM001 can be plugged into any outlet that provides 110VAC - 240VAC.

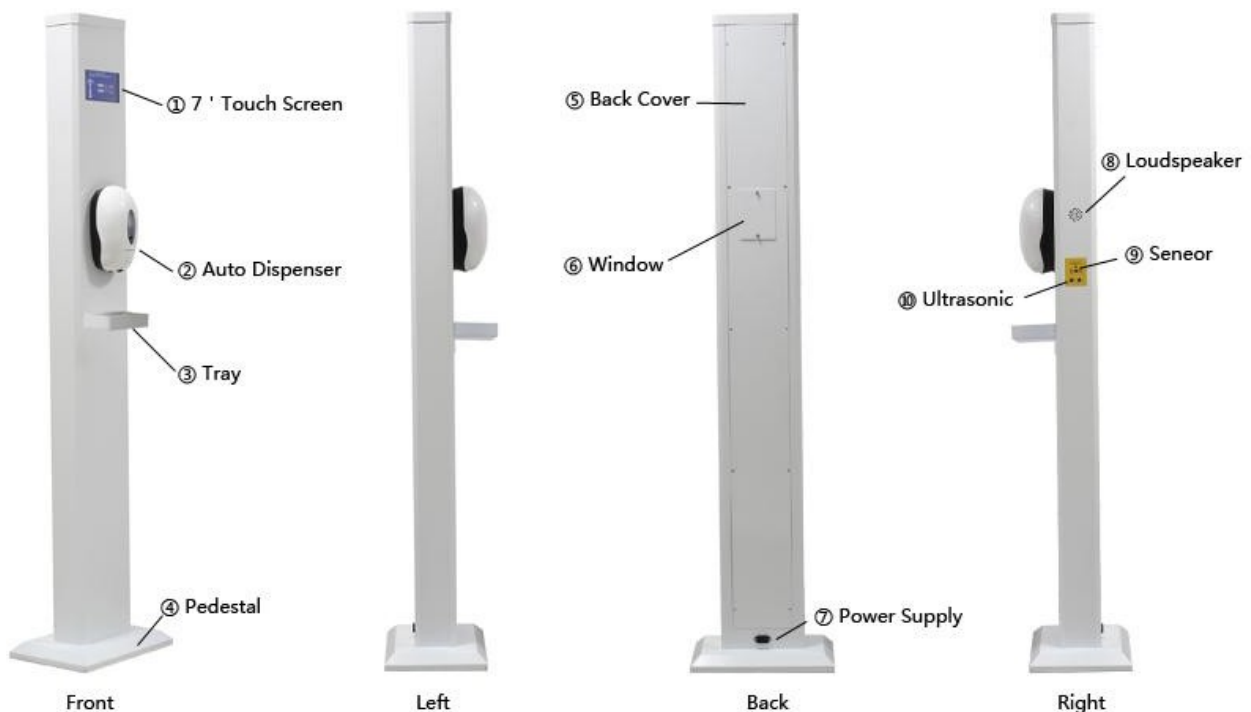

## Control Panel Settings



The ZORALM001's graphic interface is a touch screen with very simple controls. To make changes, press the "Setting" icon in the bottom right hand corner of the screen. Type in the default password of 1234, then press OK.

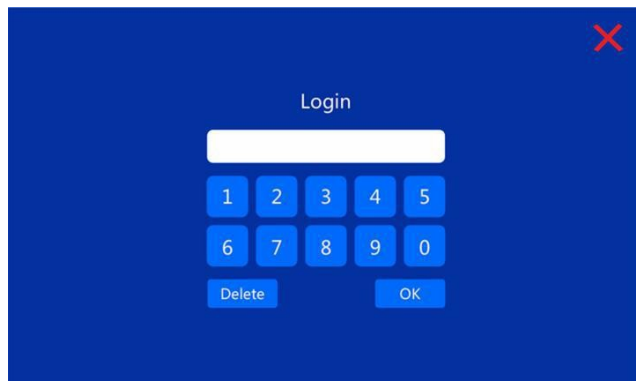

You'll be able to change the following options:

- Language: Select the language you use as your primary language. In most cases this will be selected as "English".
- Temperature: Choose either Celsius or Fahrenheit
- Voice - slide the volume bar to the appropriate setting to enable you to hear the alarms or information given.
- Data: The unit displays the number of people that pass and the number of people that fail. You can clear these counters by simply touching the X on the right hand side of the counter.
- Alarm Threshold: use the up and down arrow keys to select the temperature at which an alarm will sound. Suggested range is 2 degrees higher than a normal temperature.

After setting the variables as desired, click the confirm button at the bottom of the screen to exit the Setup Menu.

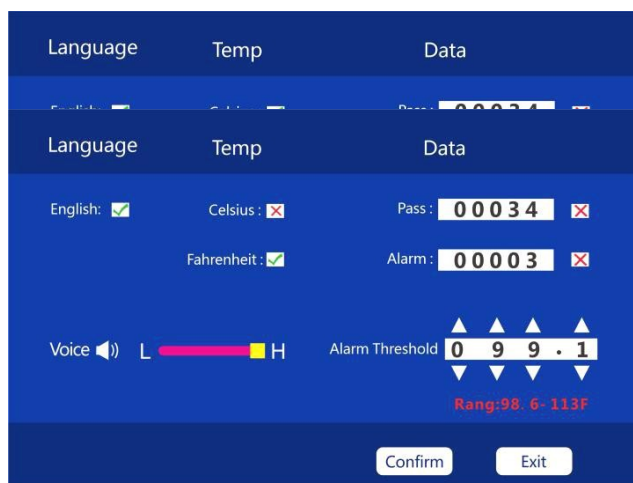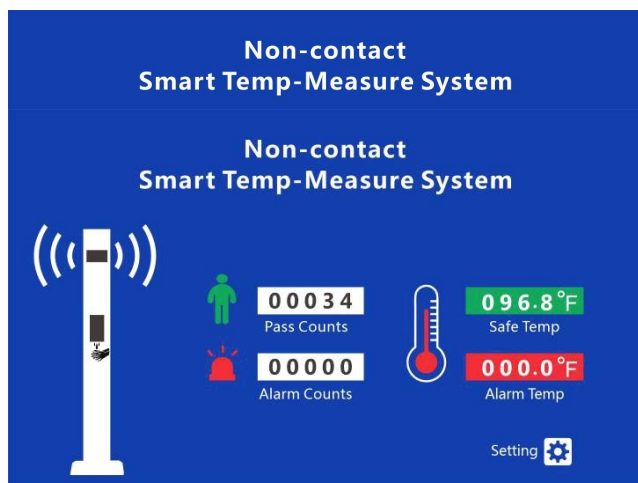

## **Hand Sanitizer**

An automated hand sanitizer dispenser is attached to the front of the ZORALM001. This convenient feature makes it easy for anyone entering your facility to make sure they have clean hands.

Simply cup your hands under the motion activated sensors near the bottom of the dispenser, and the appropriate amount of sanitizing gel will be dispensed into your hands.

Once you have the sanitizing gel in your hands, rub your hands together in a manner that evenly distributes the sanitizing gel all over each of your hands. The gel dries quickly, so there is no need to use paper towels to dry your hands.

The dispenser is re-fillable and has a capacity of 1000ML. It requires 4 AA batteries.

## **Operations**

Walk up to the unit, and hold your wrist approximately 1 - 2 inches away from the temperature sensors on the right hand side of the ZORALM001.

The screen will display the body temperature and give an audible indicator of pass or fail.

It is important to note that a secondary “medical” thermometer should be used to confirm if a person has a fever that may require additional medical treatment.

## **Calibration**

The ZORALM001 is designed for operating indoors and in a temperature controlled environment. The ZORALM001 does not require calibration. All temperature sensors are set at the factory to properly display the correct temperature. However, changing environments may lead to small shifts in how the sensor detect and display temperature. From time to time, it is advisable to compare the ZORALM001 readings to a medical thermometer. If there is a notable difference, simply change the “alarm” temperature on the ZORALM001 to appropriately reflect what you need.

## **Disclaimer**

*Our products are not used to diagnose the coronavirus. TIPTEMP is not advertising our scanners for use in the medical industry or for medical purposes. Our products can identify individuals in a population that show higher than average temperature based on*

the set parameters. Our scanners do not find individuals experiencing coronavirus symptoms. There is no way to thermally detect an infected individual who doesn't have an elevated body temperature. Only a licensed medical professional can determine if a "hot" individual is experiencing an abnormal medical condition.

## Warranty

The ZORALM001 is covered by a one year warranty against manufacture defects. All warranty replacement and repair must be approved by TIPTEMP prior to replacement or repair of the unit. Obvious abuse or neglect of the ZORALM001, will negate any warranty coverage. The warranty does not cover damage caused by dropping the unit or tipping it over on the floor or other hard surfaces.

## Technical Specifications

INFRARED BODY TEMPERATURE DETECTOR

**Model: ZORALM001 bodyTemperature SensingStand** Make the World Safer

### Product Introduction

The ZORALM001 combines a professional IPS screen and a built-in high precision infrared temperature sensor, which can quickly measure temperature in the detection area to ensure accurate and efficient temp. Users only need to place their wrist or forehead near the detection point can quickly and accurately identify their temp. And after auto-distinguishing the temp-value, it will not fluctuate with the distance, and the stability is excellent. It can be targeted at dense crowds in public places to realize non-contact measurement and screening. Personnel who encounter abnormal temp. can be detected and alarmed in time to assist the regulatory department in implementing rapid intervention and emergency measures to prevent virus epidemics in public places where there is a danger of widespread spread of infected persons.

Non-contact  
 Detection of  
 5~10cm,  
 Disinfect Hand  
 at same time

7" IPS Screen:  
 Pass Count  
 Alarm Counts  
 Safe Temp.  
 Alarm Temp.

Adjustable  
 Alarm Temp.  
 Threshold,  
 Sound/Visual  
 Alarming

Real Time  
 Broadcast  
 For Normal  
 Abnormal  
 Temp.

Export Data  
 Via Excel File  
 for Better  
 Data  
 Statistics

### Parameter

No.	Technical Standard	Technical Parameter	
1	Display	7 Inch IPS Display	
2	Sensor Height	100cm(customizable)	
3	Sensor	1 Sensor (2 optional)	
4	Data Display	Real-time display of temperature, etc.	
5	Alarm Temperature	≅ 37°C (adjustable)	
6	Test Position	Forehead and Wrist	
7	Accuracy	±0.3°C	
8	Temperature Unit	Celsius Degree/Fahrenheit(Set before leaving factory)	
8	Pass Rate	60person/minute	
9	Language	English	
10	Full Integrated Design	Easy Self installation	
11	Product Size	410(L)x1850(H)x280(W)mm	
12	Material	SPCC(Durable)	
13	Certificate	CE Approved, ISO14001 · ISO9001 · OHSAS18001	
14	N.W./G.W.	26kg/66kg	
	Packing Size	1950*440*560mm/Woodencase	
18	Working Environment	Working-Temp	-20~45°C
		Humidity	≤90%(RH)
		Working Voltage	AC100~240,50~60Hz
		Location	Indoor, Semi-outdoor, no sunlight
19	Warranty	12 month	
	Application	Train Station, School, Hospital, Office Building, etc.	

## Product Warranty, Disclaimers, Terms and Conditions, Returns

### 1. Limited Product Warranty.

1.1. Limited Warranty. TIPTEMP warrants to original Buyer that for a period of (1) year from the date of shipment of the product that such product will materially conform to specifications set forth in the Instruction Manual and be free from material defects in material and workmanship.

1.2. Disclaimer. **EXCEPT FOR THE WARRANTY SET FORTH IN SECTION 1.1 ABOVE, TIPTEMP MAKES NO WARRANTY WHATSOEVER WITH RESPECT TO THE PRODUCT, INCLUDING ANY (A) WARRANTY OF MERCHANTABILITY; OR (B) WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE; WHETHER EXPRESS OR IMPLIED BY LAW, COURSE OF DEALING, COURSE OF PERFORMANCE, USAGE OF TRADE OR OTHERWISE.**

1.3. Limitation of Remedy. Buyer's sole and exclusive remedy for any defect in the product, whether the claim arises by virtue of the limited warranty or otherwise, shall be limited to the repair or replacement (at TIPTEMP's option) of such product. If, within the warranty period, Buyer believes the product to be defective, Buyer must promptly notify TIPTEMP of the nature of the defect. The product may not be returned without first obtaining TIPTEMP's authorization and shipping instructions. If the product is defective and TIPTEMP is unable to repair the product to conform to the limited warranty, TIPTEMP will, at its option, either refund the purchase price or provide a replacement product which conforms to the limited warranty. The replaced or repaired product will have a limited warranty of 365 days from delivery of original product purchase date.

### 2. Limitation of Liability.

2.1. No Consequential or Indirect Damages. IN NO EVENT SHALL TIPTEMP BE LIABLE TO BUYER, OR ANY THIRD PARTY FOR CONSEQUENTIAL, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, PUNITIVE OR ENHANCED DAMAGES, OR LOST PROFITS OR REVENUES, ARISING OUT OF, RELATING TO, OR IN CONNECTION WITH ANY BREACH OF THIS AGREEMENT, REGARDLESS OF (A) WHETHER SUCH DAMAGES WERE FORESEEABLE, (B) WHETHER OR NOT TIPTEMP WAS ADVISED OF THE POSSIBILITY OF SUCH DAMAGES AND (C) THE LEGAL OR EQUITABLE THEORY (CONTRACT, TORT OR OTHERWISE) UPON WHICH THE CLAIM IS BASED.

2.2. Maximum Liability. IN NO EVENT SHALL ZORPRO'S LIABILITY ARISING OUT OF OR RELATED TO THIS AGREEMENT, WHETHER ARISING OUT OF OR RELATED TO BREACH OF CONTRACT, TORT (INCLUDING

NEGLIGENCE) OR OTHERWISE, EXCEED THE TOTAL OF THE AMOUNT PAID TO TIPTEMP FOR THE PRODUCT.

Buyer acknowledges and agrees that the parties entered into the agreement for the sale and purchase of the product in reliance upon the limitations of liability set forth in this Section 2, that the same reflect an allocation of risk between the parties (including the risk that a contract remedy may fail of its essential purpose and cause consequential loss), and that the same form an essential basis of the bargain between the parties.

3. Governing Law and Forum. This Agreement and all documents related to the product, and all matters arising out of or relating to this Agreement, whether sounding in contract, tort, or statute, are governed by, and construed in accordance with, the laws of the State of Utah, United States of America, without giving effect to the conflict of laws provisions thereof to the extent such principles or rules would require or permit the application of the laws of any jurisdiction other than those of the State of Utah. Buyer irrevocably consents to the exclusive jurisdiction of the Fourth District Court, State of Utah, in and for Utah County, regarding all controversies between the parties.
4. Modification. None of these terms and conditions may be modified, waived or discharged unless such waiver, modification or discharge is agreed to in writing signed by both the Buyer and TIPTEMP.
5. Entire Agreement. These terms and conditions constitute the entire agreement between the parties and supersede any prior proposals, advertisements, negotiations, understandings, agreements or representations.

### Return Policy:

We do not accept any returns on products sold. If an item sold by TIPTEMP is under warranty we will fix or replace the broken item.